



**COLLECTION AND SHIPMENT OF BLOOD SAMPLES FOR IFAR REGISTRY  
THE ROCKEFELLER UNIVERSITY HOSPITAL  
Laboratory of Genome Maintenance  
Agata Smogorzewska, MD, PhD                      Phone: 212-327-7850**

**PURPOSE:**

This sample is being obtained **for research purposes**. Any genetic testing that might be done on this sample has to be confirmed in a clinical laboratory. As the sample is being obtained for research purposes, we cannot say how long it will take to discover findings. Sometimes it can take months or years before we are able to provide any information to the referring physician.

**CONSENT:**

No blood should be collected unless a valid consent is obtained. For participants already enrolled in IFAR, the consent may already be on file. Please check with Jennifer Kennedy at 212-327-8612 or Erica Sanborn at 212-327-8613 if there are any questions about the consent status.

**COLLECTION:**

Peripheral blood should be drawn into 10cc sodium heparin (GREEN TOP TUBES). For adults and pediatric patients over age 5, we require 10cc. For newborns we would like at least 3cc. For all other pediatric patients we need at least 5cc

**PACKING:**

It is extremely important to carefully pack the tube(s), providing ample protection against breakage. Use a watertight container such as a cylindrical can or Styrofoam box made for shipping blood tubes along with absorbent material. This can then be shipped in a Federal Express Clinical Pak or other sturdy outer packaging. FedEx air bills can be provided by IFAR, but please complete Sender's Info before shipping.

**TEMPERATURE:**

Specimens should be sent at room temperature. Do not ship on ice or dry ice. Do not freeze.

**SHIPPING:**

SEND VIA FEDERAL EXPRESS FOR PRIORITY OVERNIGHT DELIVERY to:  
The Rockefeller University Laboratory of Genome Maintenance Attn: Frank Lach  
1230 York Avenue New York, NY 10065  
Notify the laboratory by calling (212) 327-8862, prior to shipping

**REQUISITION:**

The physician is asked to provide patient information on the attached requisition form, which should accompany the sample.

**TIME:**

Samples should be drawn/shipped MONDAY-THURSDAY (to arrive at our lab Tuesday-Friday). Samples should NOT be drawn/shipped on Friday, Saturday or Sunday.

**QUESTIONS/CONTACT INFORMATION:**

Questions about the research study contact one of our Genetic Counselors:

Jennifer Kennedy at 212-327-8612 or Erica Sanborn at 212-327-8613.

Questions about specimen retrieval/shipment contact Frank Lach at 212-327-8862